

Take a closer look.



Cellvizio®

With the Cellvizio system, you can visualize tissue at the cellular level in vivo during ureteroscopic and cystoscopic procedures.




Cellvizio is a registered trademark of Mauna Kea Technologies.


Cellvizio®

See into the future of in vivo imaging.



Cellvizio is a probe-based confocal laser endomicroscopy (pCLE) system. The system is based on a laser scanning technology that was adapted for imaging through a bundle of optical fibers. These fiber bundles are assembled with other optical elements to form the Confocal Miniprobes™.

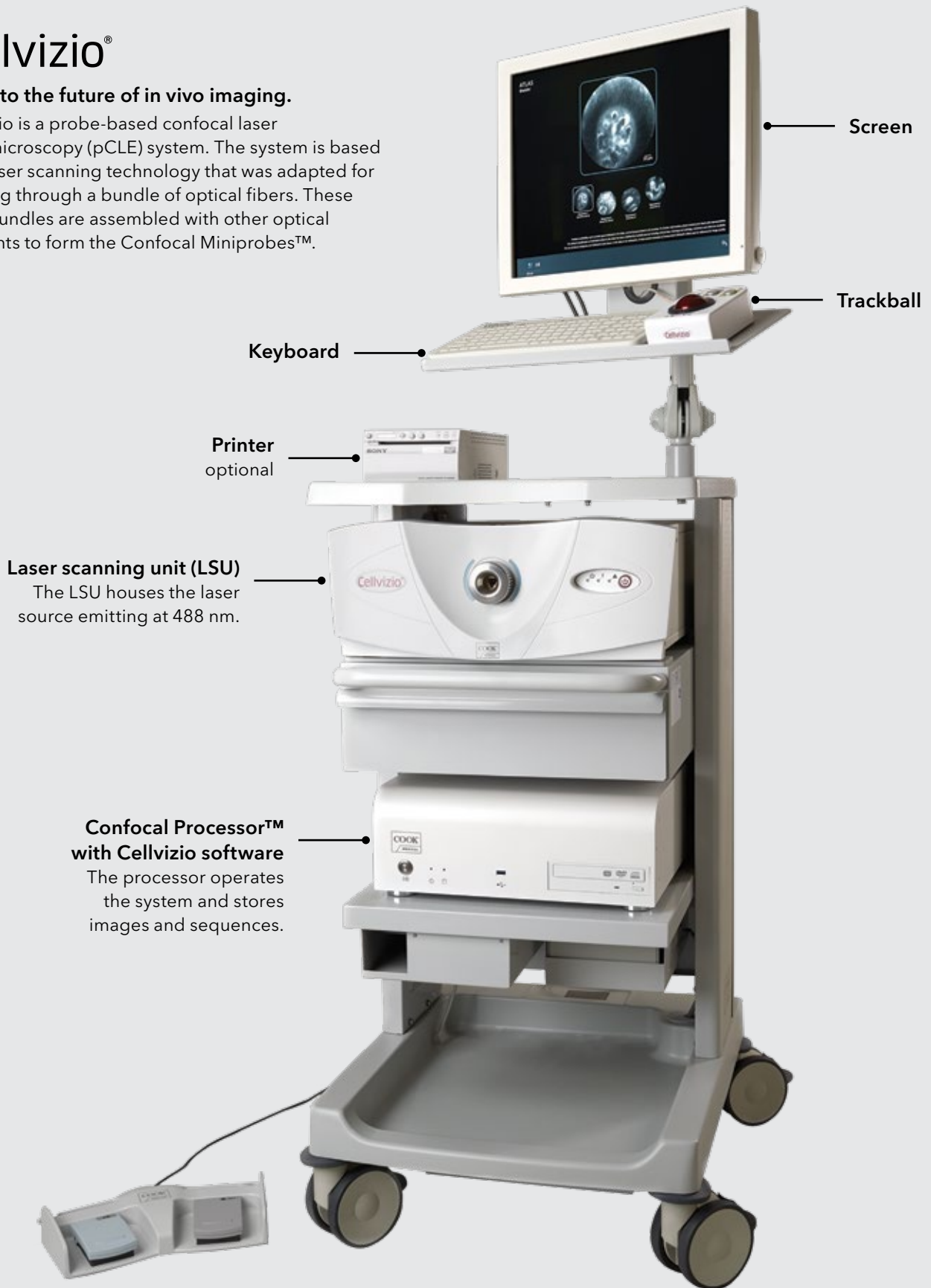

Endoscopic image

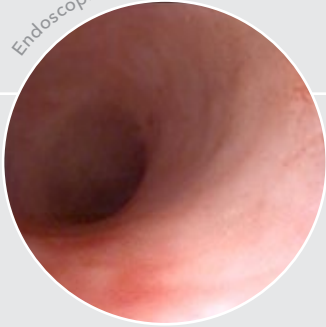

Cellvizio image

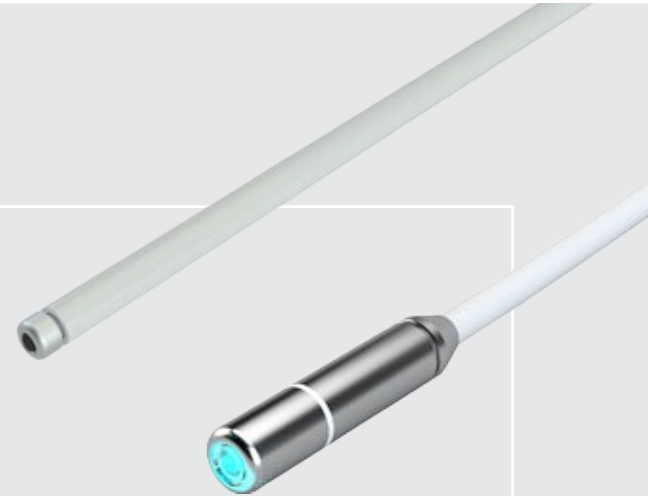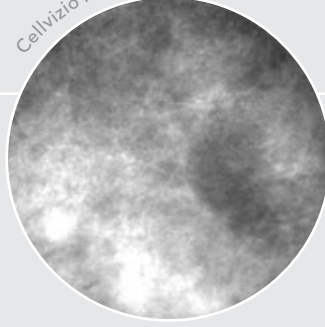

Confocal Miniprobes

The reusable miniprobes are exclusively for use with the Cellvizio system. They connect to the LSU, transport the laser beam to the site of observation, and capture the fluorescent light emitted back from the tissue. This allows for real-time cellular imaging.

The miniprobes have a limited number of uses. They must be reprocessed prior to their first use and immediately after each procedure, according to the reprocessing instructions provided.

Name	Fr	Length m	Maximum field of view µm	Depth of observation µm	Number of uses	Intended use
UroFlex™ B	3.0	3	Ø 325	40-70	10	Cystoscopy and ureteroscopy
CystoFlex™ F	3.0	2	Ø 325	40-70	20	Flexible cystoscopy
CystoFlex UHD™ R	8.4	2	Ø 240	55-65	20	Rigid cystoscopy

Order Number	Reference Part Number	Fr	Length m	Description
Cellvizio system				
G47269	CS-100-488-CK-1	-	-	North American plug configuration
Confocal Miniprobes				
G47273	MP-0011-URF-CK	3.0	3	reusable up to 10 times
G47274	MP-0012-CYS-CK	3.0	2	reusable up to 20 times
G47275	MP-0013-HCYS-CK	8.4	2	reusable up to 20 times

UroFlex, CystoFlex, and CystoFlex UHD are trademarks of Mauna Kea Technologies.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Customer Service

EU Website: cookmedical.eu

EDI: cookmedical.eu/edi

Distributors: +353 61239240, ssc.distributors@cookmedical.com

Austria: +43 179567121, oe.orders@cookmedical.com

Belgium: +32 27001633, be.orders@cookmedical.com

Denmark: +45 38487607, da.orders@cookmedical.com

Finland: +358 972519996, fi.orders@cookmedical.com

France: +33 171230269, fr.orders@cookmedical.com

Germany: +49 6950072804, de.orders@cookmedical.com

Ireland: +353 61239252, ie.orders@cookmedical.com

Italy: +39 0269682853, it.orders@cookmedical.com

Netherlands: +31 202013367, nl.orders@cookmedical.com

Poland: +48 223060159, pl.orders@cookmedical.com

Spain: +34 912702691, es.orders@cookmedical.com

Sweden: +46 858769468, se.orders@cookmedical.com

Switzerland - French: +41 448009609, fr.orders@cookmedical.com

Switzerland - Italian: +41 448009609, it.orders@cookmedical.com

Switzerland - German: +41 448009609, de.orders@cookmedical.com

United Kingdom: +44 2073654183, uk.orders@cookmedical.com

USA Website: cookmedical.com

EDI: cookmedical.com/edi.do

Americas:

Phone: +1 812.339.2235, 855.207.7214, Fax: 800.330.5490

E-mail: orders@cookmedical.com

Australia:

Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283

E-mail: cau.custserv@cookmedical.com



URO-CAP-EQ-85X11